



BLOXR

Warranty and Instructions for Use Radiation Protection Glasses Effective: September 12, 2025

Radiation Protection Glasses

For scatter radiation protection during fluoroscopic procedures.

Warranty:

- Eyeglasses will be free of defects in material and workmanship for one year from date of purchase.
- Warranty does NOT cover damage caused by dropping, abuse, negligence or unauthorized repairs. Warranty does not cover scratching of the lenses.
- Warranty is not transferable to another wearer.

Prescriptions:

- Any problem with your prescription must be communicated to BLOXR Solutions within **15 days** of receipt.
- Prescriptions are not covered by the warranty.

Usage:

- It is recommended to secure the eyewear strap when wearing eyeglasses to prevent dropping.
- Keep glasses inside the protective case when not in use.

Cleaning:

- Do not subject lenses to abrasive or gritty materials.
- Use a lint free cleaning cloth or protective silk sleeve to wipe lenses in a straight line when cleaning (not in a circular motion).
- When handling eyewear during cleaning, be careful not to scratch or drop the eyeglasses.

Caution:

- This leaded eyewear is specially designed to protect against scatter radiation during fluoroscopic procedures. Lenses provide 0.75mm Pb protection and side shields (if included) provide 0.5mm Pb protection. In use, the specific level and/or amount of radiation protection will vary depending on the fluoroscopic energy levels utilized with each procedure.
- Lenses are not shatterproof; this eyewear is not intended to be used in place of safety glasses.

If you have questions or concerns
about your eyewear, please contact:

BLOXR Solutions, 2250 S. 1300 West, Suite C, Salt Lake City, UT 84119 USA
855-256-9729, www.bloxr.com
Customer Service email: order@bloxr.com